

H&E (HEMATOXYLIN & EOSIN) STAIN



METHOD – STAIN

PRODUCT CODE – ST28

INSTRUCTIONS FOR USE

INTENDED USE: HISTOPATHOLOGY STAIN.

SUMMARY

Hematoxylin and Eosin (H & E) staining is the most common staining technique in histopathology. This uses a combination of two dyes, Hematoxylin and Eosin used for demonstration of nucleus and cytoplasmic inclusions in clinical specimens.

METHOD PRINCIPLE

Alum acts as mordant and hematoxylin containing alum stains the nucleus light blue. This turns red in presence of acid, as differentiation is achieved by treating the tissue with acid solution. Bluing step converts the initial soluble red color within the nucleus to an insoluble blue color. The counterstaining is done by using eosin which imparts pink color to the cytoplasm.

REAGENTS

Haematoxylin(Harris)	Haematoxylin, Potassium alum and Mercuric oxide.
Eosin	Eosin ,Buffer
Dehydrant	Denatured Alcohol

PRECAUTIONS

This product is for in Vitro diagnostics use and should be used by properly trained individuals. Precautions should be taken against the dangers of microbiological hazards by properly sterilizing specimens, containers and media after use. Directions should be read and followed carefully.

STORAGE

Store product in its original container at room temperature (<30 °C) until expiry. Keep container tightly closed during storage. Avoid exposure to bright light.

PRODUCT DETERIORATION

This product should not be used if:

1. The colour has changed.
2. The expiration date has passed.
3. There are other signs of deterioration.

MATERIALS REQUIRED BUT NOT SUPPLIED

1. Microscopic Slides.
2. Microscope.
3. DPX mountant.
4. Spirit Lamp
5. Coplin Jars with Lids

SPECIMEN COLLECTION

Clinical Samples – Blood Smear, Body Fluids, , Soft tissue, Normal tissue

For clinical samples follow appropriate techniques for handling specimens as per established guidelines.

REAGENT PREPARATION:

Reagent are ready- to- use.

PROCEDURE

1. Bring the section to slide using suitable adhering agent.
2. Slightly warm to melt the wax .
3. Remove the wax with Xylene by giving 3 times washing.
4. Remove the Xylene in 3 times washing with Dehydrant.
5. Dip the smear into distilled water for 3 minutes.
6. Dip the smear in Haematoxylin (Harris) stain for 1min.
7. Wash with Distilled water.
8. Dip the smear in Eosin stain for 60 seconds.
9. Wash with distilled water for 15 seconds.
10. Rinse with Dehydrant and Xylene till it gets cleared.
11. Mount with glass cover using a drop of DPX .

RESULTS AND INTERPRETATION

Refer to appropriate references for appearance of cytoplasm or differential characteristics of Papnicolau stained smears.

RESULTS AND INTERPRETATION

Nuclei	: Blue and Black
Cytoplasm	: Pink
Muscle Fibres	: Deep Red
RBCs	: Orange Red
Fibrin	: Deep Red

QUALITY CONTROL

All lot numbers of H&E have been tested and found to be acceptable. The patient smear can serve as quality control to verify the efficacy of the staining reagents. A smear made from a patient specimen (previously identified as positive) with at least one field may also be included to verify differential staining characteristics and compare with specimen stain results. If aberrant quality control results are noted, patient results should not be reported.

BIBLIOGRAPHY

1. Microlab-inc.com/reagents-stains
2. Carson, Freida L; Hladik, Christa (2009). Histotechnology: A Self-Instructional Text (3 ed.). Hong Kong: American Society for Clinical Pathology Press. pp. 361–3363. ISBN 978-0-89189-581-7.

SYMBOLS:



Read Instruction for use



In Vitro Diagnostic Use Only



Manufactured by



Expiry Date



Storage Temperature

ANAMOL LABORATORIES PVT. LTD.

61, Genesis Industrial Township, Kolgaon,
Palghar – 401 404. India.

Customer Care & WhatsApp: +91-9823388695.

admin@anamollabs.com

exports@anamollabs.com

www.anamollabs.com

ISO 9001 : 2015

ISO 13485 : 2003

GMP

CE